



TriMed Hand Plating System



# Hand Plates

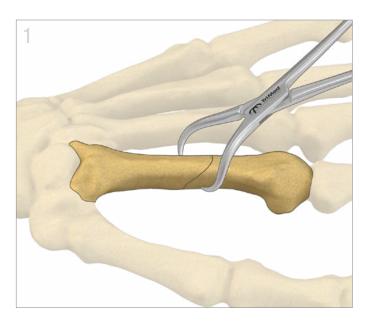

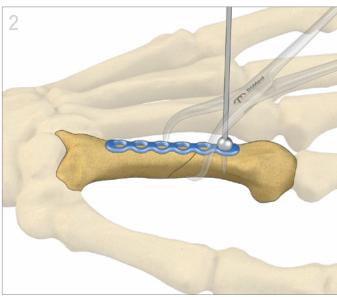



## **Exposure and Temporary Fixation**

- Expose the metacarpal or phalanx with standard techniques.
- Reduce the fracture or osteotomy and hold with K-wires or a bone reduction clamp.

## **Plate Placement**

- Position plate over the fractured or osteotomized bone.
- Temporarily secure plate to the bone using plate tacks.


#### **Screw Insertion**

- Use the appropriate drill guide and corresponding drill for the desired screw size. (Reference Screw Table on last page)
- Verify screw length with corresponding depth gauge. Insert screw into plate.

**Note:** When using locking drill guides ensure installment and placement is concentric to the screw hole. Off-axis guide placement can result in screws not locking into the plate; locking screws can only be used on-axis.

# Hand Plates



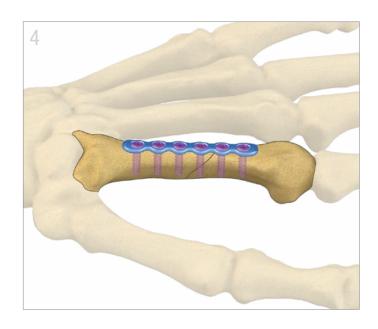


## **Final Fixation**

• Complete fixation with additional screws as necessary. Check final screw placement and reduction.


#### TIPS

## **Interfragmentary Fixation**


• Drill hole with appropriate drill and drill guide. Measure and insert screw. Countersink as needed.

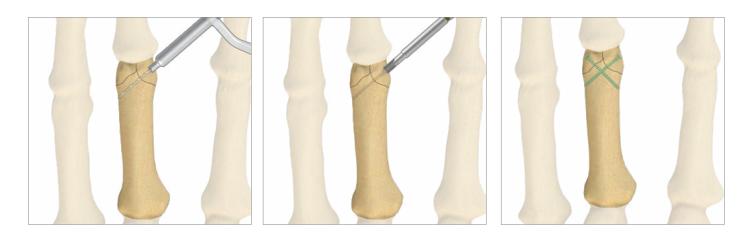


### **Plate Cutter**

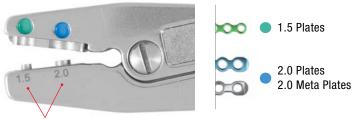



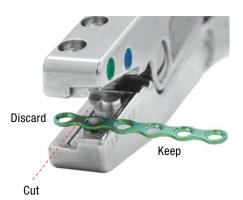

All implants made from surgical grade titanium

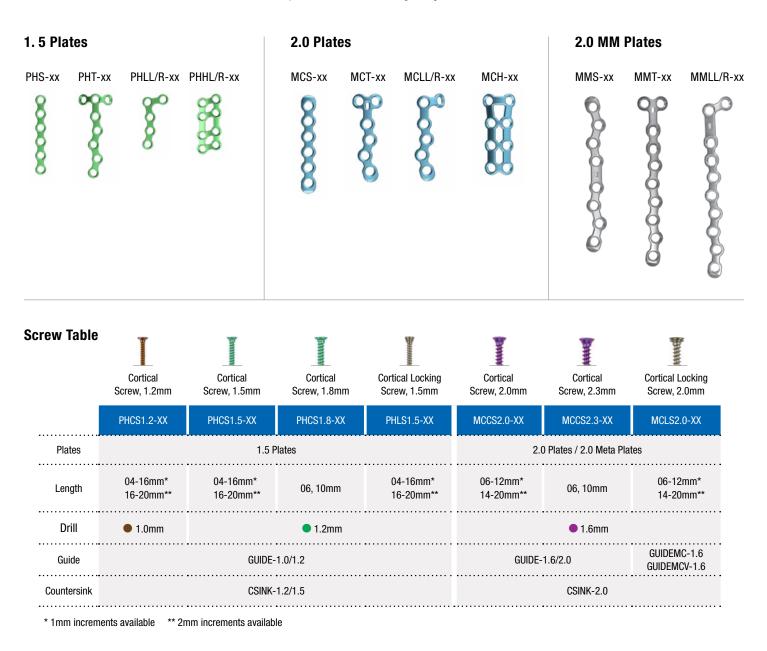



TriMed, Inc. / 27533 Avenue Hopkins / Santa Clarita, CA 91355 USA / 800-633-7221 / www.trimedortho.com

The technique presented is one suggested surgical technique. The decision to use a specific implant and the surgical technique must be based on sound medical judgment by the surgeon that takes into consideration factors such as the circumstances and configuration of the injury. This document is controlled by TriMed. Inc. When downloaded, printed, and/or cooied, this document becomes uncontrolled, and users should always check trimedortho.com for the latest version.

For indications, contraindications, warnings and precautions related to TriMed Hand Plating System reference IFU on trimedortho.com/ifu.

See trimedortho.com/patents for all patent information.