




# Small Threaded IM Nails


Surgical Technique | *TriMed Threaded IM Nail System*



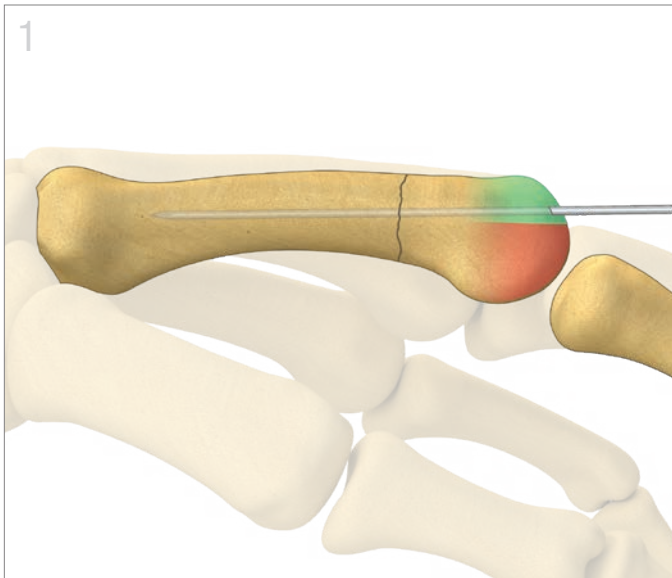

## K-wire Placement


- Expose metacarpal/phalangeal head to access the dorsal third of the articular surface by retracting or splitting extensor tendon.
- Select the appropriate K-wire for the desired size Small Threaded IM Nail.
- Reduce fracture and direct a K-wire central to the dorsal third. Drive K-wire with consideration for the curvature of the volar wall and overall screw trajectory. Confirm with C-arm.

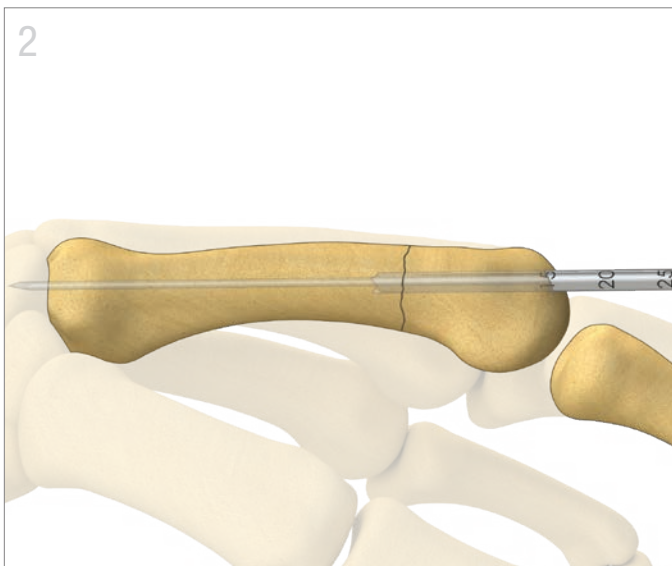

## Measure and Ream

- With the K-wire positioned to the desired length, measure the length with the Wire Gauge. After measuring, advance the K-wire into the opposite bone end to prevent inadvertent removal when reaming.
- Ream over K-wire to desired depth, crossing the fracture site.<sup>1</sup>
- Use laser markings on drill as an aid to select appropriate nail length.

<sup>1</sup> **Note:** Radiographic measurements and intramedullary canal reaming should be used to size the canal appropriately prior to implant insertion.

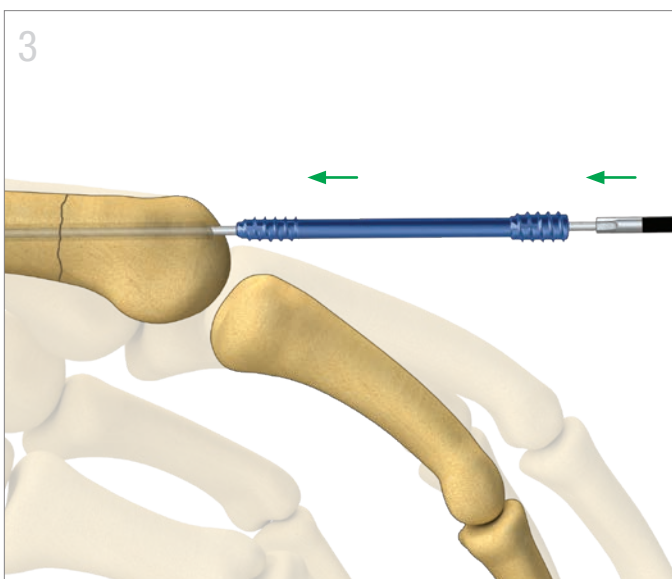


## Small Threaded IM Nail Placement

- Guide the appropriate length Small Threaded IM Nail over the K-wire.
- Engage the appropriate driver and advance nail by threading into the canal to desired position. The head of the nail should be recessed below the subchondral surface.




4

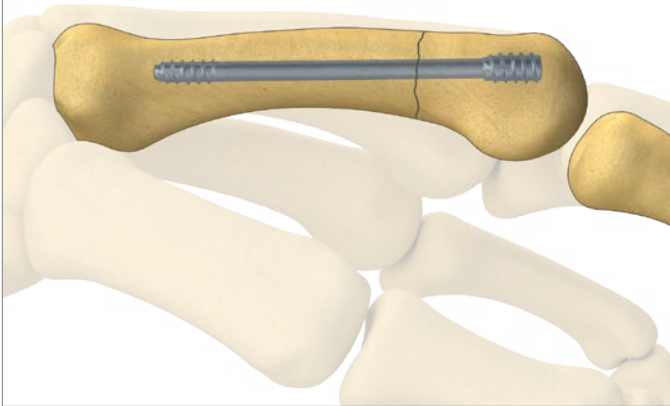

## Final Fixation

- Confirm position, depth, and placement with C-arm.
- Remove K-wire.

## REMOVAL TOOL

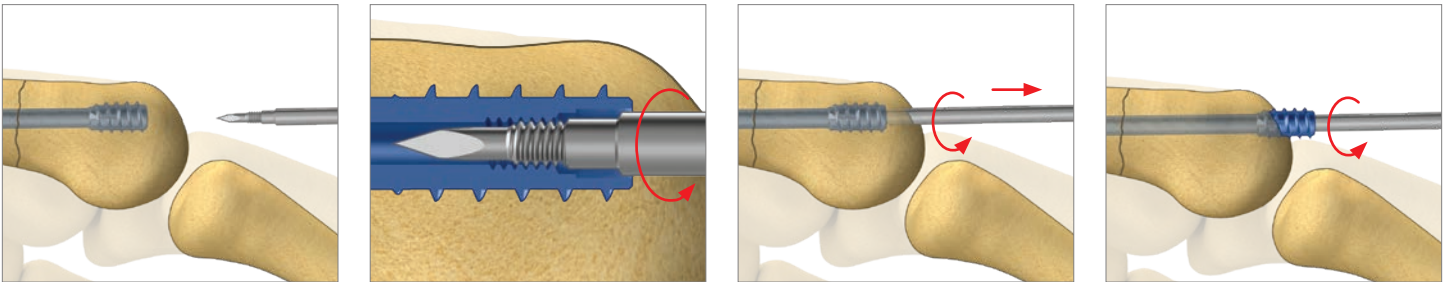



## Small Threaded IM Nail Removal

- If done post-operatively, use the appropriately sized K-wire to find the cannulation in the Small Threaded IM Nail. If necessary, ream the channel above the head carefully with the drill over the K-wire.
- Remove the K-wire and insert Removal Tool to engage the socket opening at the head of the implant.
- Rotate the Removal Tool **counter-clockwise** to engage and lock the reverse thread within the implant.
- Continue to rotate counter-clockwise while withdrawing the Removal Tool to unscrew the implant from the bone.

**Note:** Excessive force may cause instrument breakage or damage. If excessive torque is required, another method for implant removal will be needed. The Removal Tool is a single-use device.

All implants made from surgical grade titanium alloy

## Small Threaded IM Nails

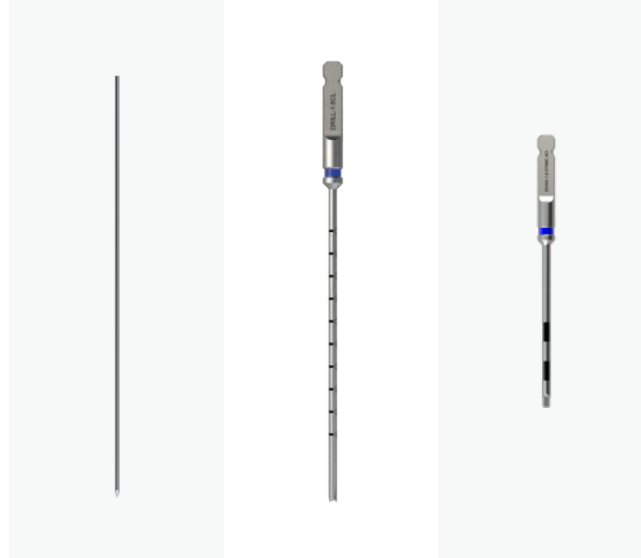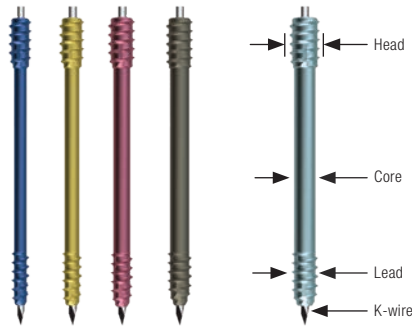

Screw	Lengths	Core	K-wire	Drill Bit	Driver	Removal Tool
<b>1.8</b> IMN-1.8-xx T 	14-50mm	1.8mm	WIRE-0.9/120	DRILL-1.8/100C S	DVHX-1.6/070MC AO	RMVIMN-1.8/2.1
<b>2.1</b> IMN-2.1-xx T 	20-60mm	2.1mm	WIRE-1.1/120	DRILL-2.1/100C S	DVHX-1.8/070MC AO	RMVIMN-1.8/2.1
<b>2.4</b> IMN-2.4-xx T 	30-70mm	2.4mm	WIRE-1.1/120	DRILL-2.4/110C S	DVHX-2.0/070MC AO	RMVIMN-2.4/2.7
<b>2.7</b> IMN-2.7-xx T 	30-70mm	2.7mm	WIRE-1.1/120	DRILL-2.7/110C S	DVHX-2.0/070MC AO	RMVIMN-2.4/2.7
<b>3.0</b> IMN-3.0-xx T 	30-70mm	3.0mm	WIRE-1.1/120	DRILL-3.0/110C S	DVHX-2.4/070MC AO	RMVIMN-3.0

**Note:** Not all lengths listed above are provided in the standard tray/system configuration. May be subject to special order request.



TriMed, Inc. / 27533 Avenue Hopkins / Santa Clarita, CA 91355 USA / 800-633-7221 / [www.trimedortho.com](http://www.trimedortho.com)

The technique presented is one suggested surgical technique. The decision to use a specific implant and the surgical technique must be based on sound medical judgment by the surgeon that takes into consideration factors such as the circumstances and configuration of the injury.

This document is controlled by TriMed, Inc. When downloaded, printed, and/or copied, this document becomes uncontrolled, and users should always check [trimedortho.com](http://trimedortho.com) for the latest version.

For indications, contraindications, warnings and precautions related to TriMed Threaded IM Nail System reference IFU on [trimedortho.com/ifu](http://trimedortho.com/ifu).

See [trimedortho.com/patents](http://trimedortho.com/patents) for all patent information.