



# Bridge Plate


Surgical Technique | *TriMed Wrist Fixation System*






## Exposure

- Perform closed reduction with traction.
- Make an incision over the midshaft of either the 2<sup>nd</sup> or 3<sup>rd</sup> metacarpal.
- A second incision is made just proximal to the thumb outcropping muscles. Expose the radial shaft.
- An optional 3<sup>rd</sup> incision can be made over Lister's to mobilize the EPL or to create a graft portal.

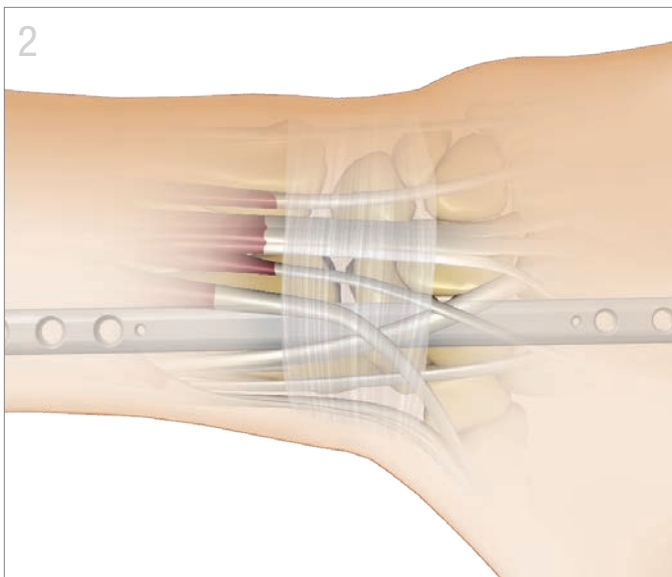


## Plate Insertion

- Using the Bridge Plate Inserter, insert plate from proximal to distal, or from distal to proximal.
- Pass the plate under the extensor tendons along the surface of the bone until it appears through the opposite incision.

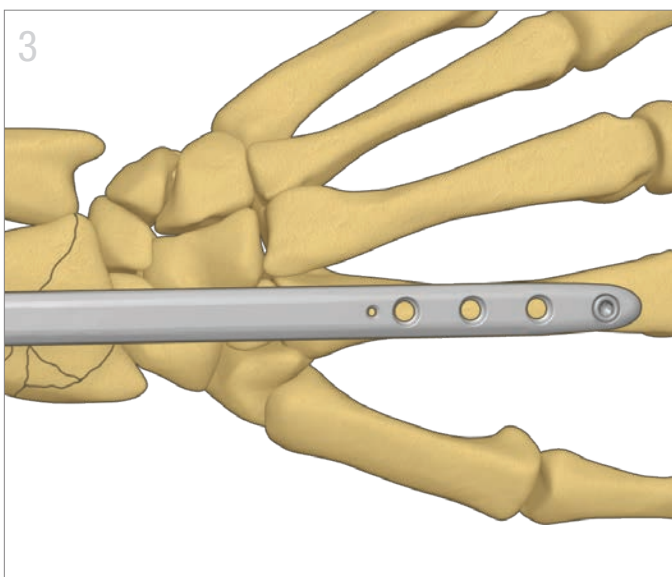


## Provisional Distal Fixation

- Position the plate over the desired metacarpal to allow placement of distal 2.7mm screws.
- Place a non-locking 2.7mm screw using a 2.0mm (yellow) drill in the most distal hole to lag the plate to bone.

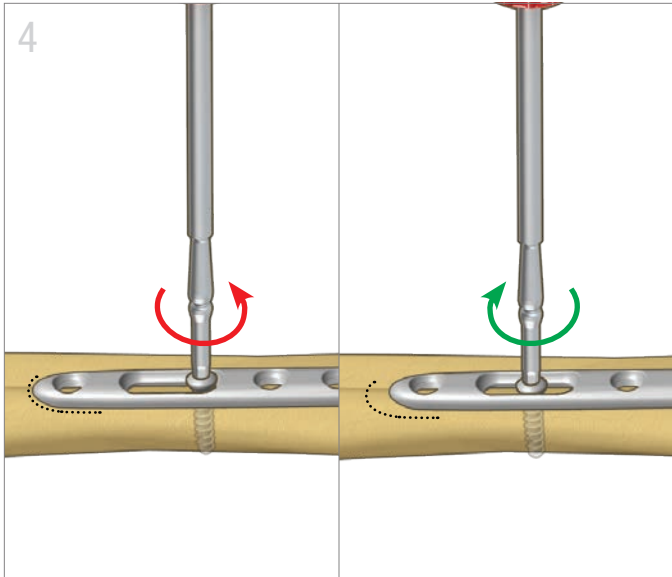

## Secure Plate and Restore Length

- With the wrist in neutral rotation, center the plate over the metacarpal and the radial shaft.
- Place a non-locking 3.2mm screw using a 2.3mm (red) drill into the distal end of the slotted hole.
- To gain additional length, loosen screw 1/4 turn, apply traction and retighten screw.

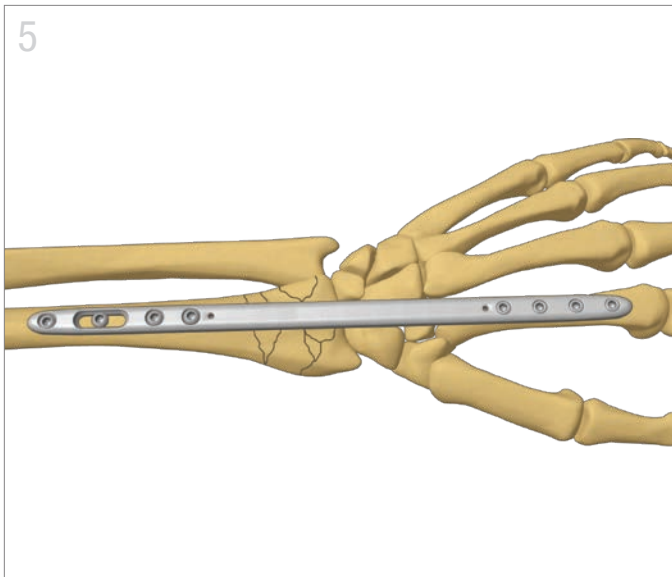

## Final Fixation

- Secure the plate by placing additional screws
  - 2.7mm locking or non-locking screws distally
  - 3.2mm locking or non-locking screws proximally





**Note:** When using locking drill guides or quick guides ensure installment and placement is concentric to the screw hole. Off-axis guide placement can result in screws not locking into the plate; locking screws can only be used on-axis.

## TIPS

- Displaced or unstable volar ulnar corner, dorsal ulnar corner, or die-punch fragments may require augmented fixation.
- Buttress Pins™, Hook Plates™, or Pin Plates™ can be used as adjunct fixation to a Bridge Plate.



All implants made from surgical grade stainless steel

Screw Table				
	Cortical Screw, 2.7mm	Locking Screw, 2.7mm	Cortical Screw, 3.2mm	Locking Screw, 3.2mm
	HEX2.7-XX	LCBS2.7-XX	HEX3.2-XX	LHEX3.2-XX
Length	08-18mm*	08-18mm*	08-20mm* 11-15mm**	10-20mm*
Drill	● 2.0mm		● 2.3mm	
Guide	GUIDE-2.0/2.7	GUIDELCBS-2.0	GUIDE-2.3/3.2	GUIDEQ-2.3
Driver	2.0mm HEX		2.5mm HEX	

\* 2mm increments \*\* 1mm increments

## Bridge Plate

BRGP



## Bridge Plate Inserter

INSRTR-BRG



## Quick Guide

GUIDEQ-2.3



## Drill Guide

GUIDELCBS-2.0



TriMed, Inc. / 27533 Avenue Hopkins / Santa Clarita, CA 91355 USA / 800-633-7221 / [www.trimedortho.com](http://www.trimedortho.com)

The technique presented is one suggested surgical technique. The decision to use a specific implant and the surgical technique must be based on sound medical judgment by the surgeon that takes into consideration factors such as the circumstances and configuration of the injury.

This document is controlled by TriMed, Inc. When downloaded, printed, and/or copied, this document becomes uncontrolled, and users should always check [trimedortho.com](http://trimedortho.com) for the latest version.

For indications, contraindications, warnings and precautions related to TriMed Wrist Fixation System reference IFU on [trimedortho.com/ifu](http://trimedortho.com/ifu).

See [trimedortho.com/patents](http://trimedortho.com/patents) for all patent information.

CE  
1639