



Threaded Titanium Suture Anchor


2, 3, 4, 6mm

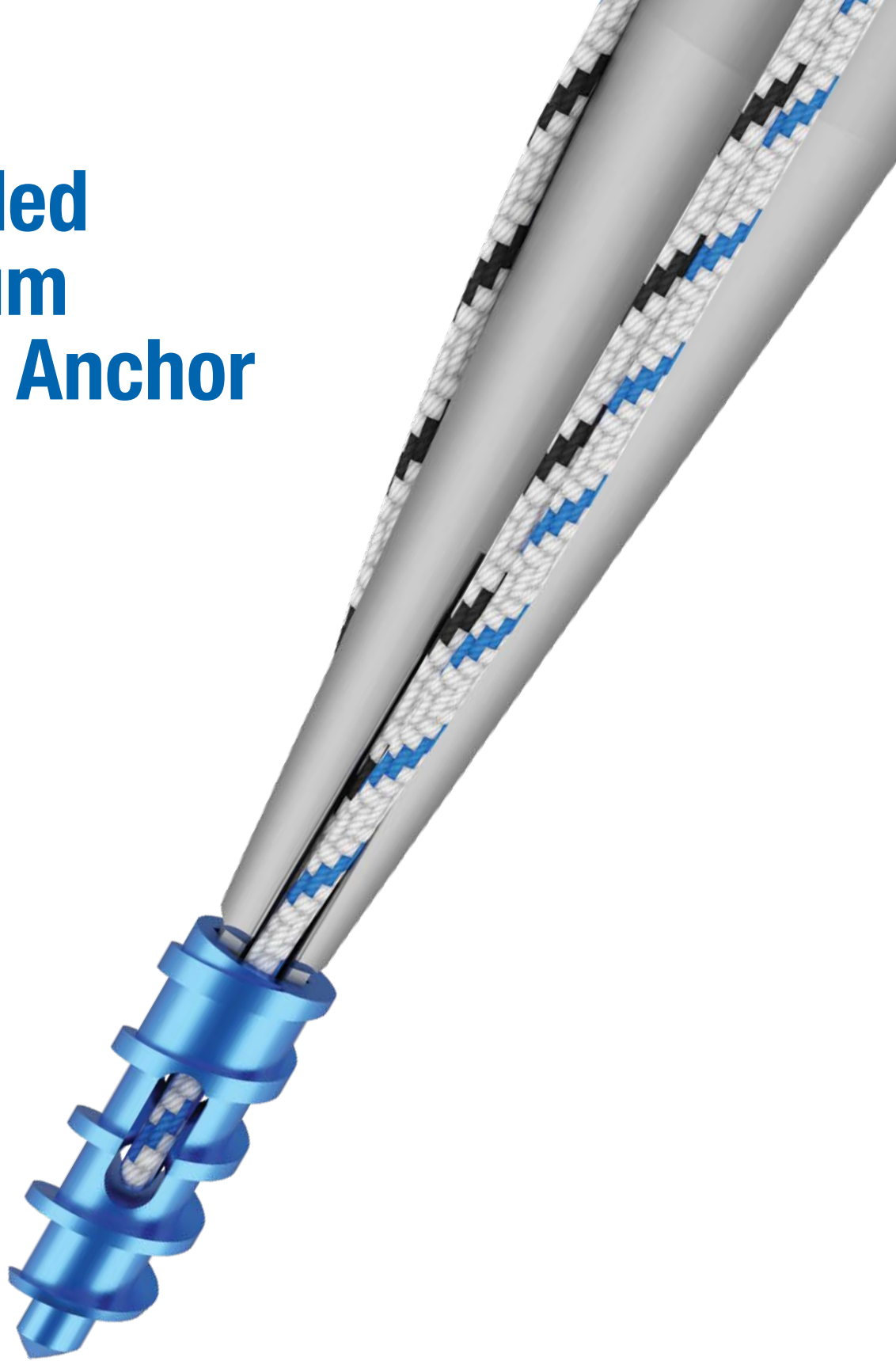

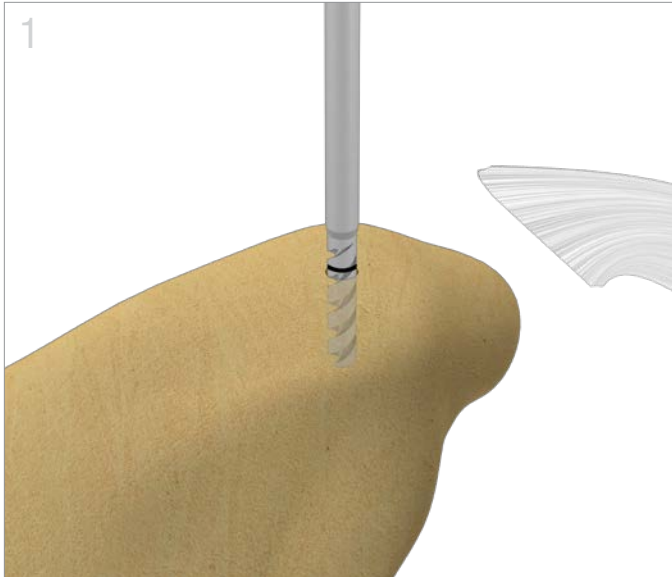

Site Preparation

- Prepare the bone and tendon/ligament at the site of soft tissue repair.
- Drill a hole at the site of attachment to the depth of the mark on the drill.

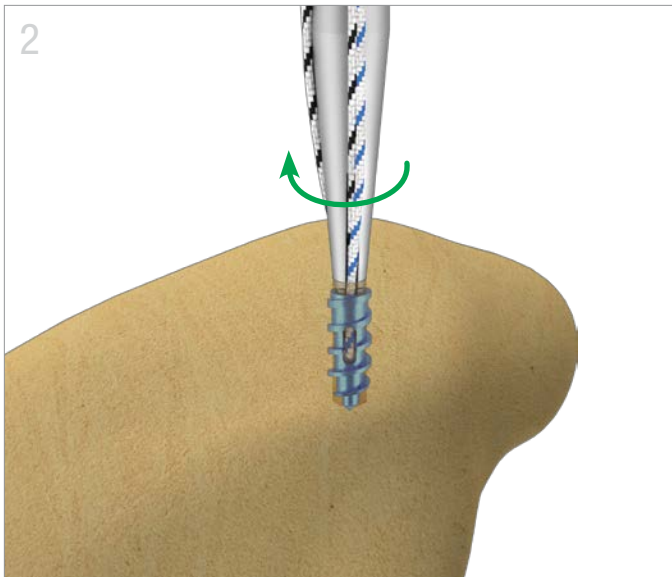

Anchor Insertion


- Screw in anchor until the top of the anchor is flush with the surface of the bone.



Inserter Disengagement

- Slide the cover off the inserter.
- Remove the needles and suture from the inserter, two needles at a time.¹
- Disengage inserter from anchor.

¹ **Note:** Do not release all 4 needles at the same time

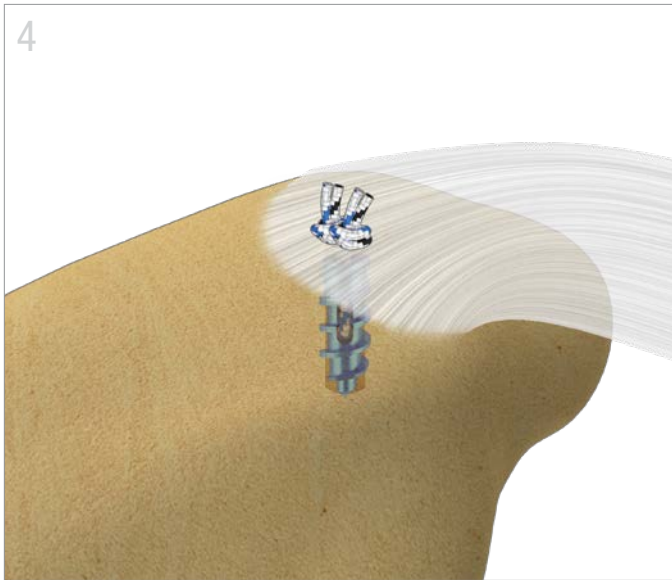

Final Fixation

- Tighten and tie the sutures to secure tendon/ligament against the bone.

Indications²


	2mm	3, 4, 6mm
Upper Extremity		
Hand / Wrist	●	●
Elbow	●	●
Shoulder		●
Lower Extremity		
Foot / Ankle	●	●
Knee		●
Hip		●

Anchor is made from surgical grade titanium. Sutures are UHMWPE.

Anchor Configuration

Catalog #	ATT-2x6 2S ND	ATT-3x11 2S N	ATT-4x13 2S N	ATT-6x17 2S N
Size (mm)	2 x 6	3 x 11	4 x 13	6 x 17
Sterile Anchor	Suture (qty / USP)	2ea / #3-0	2ea / #2	2ea / #2
	Tape (qty / size)	-	-	-
	Needles (qty)	4	4	4
	Drill	Included	2.0/110mm	2.5/110mm
Non Sterile Tools	Tap	-	-	-

Anchor Assembly

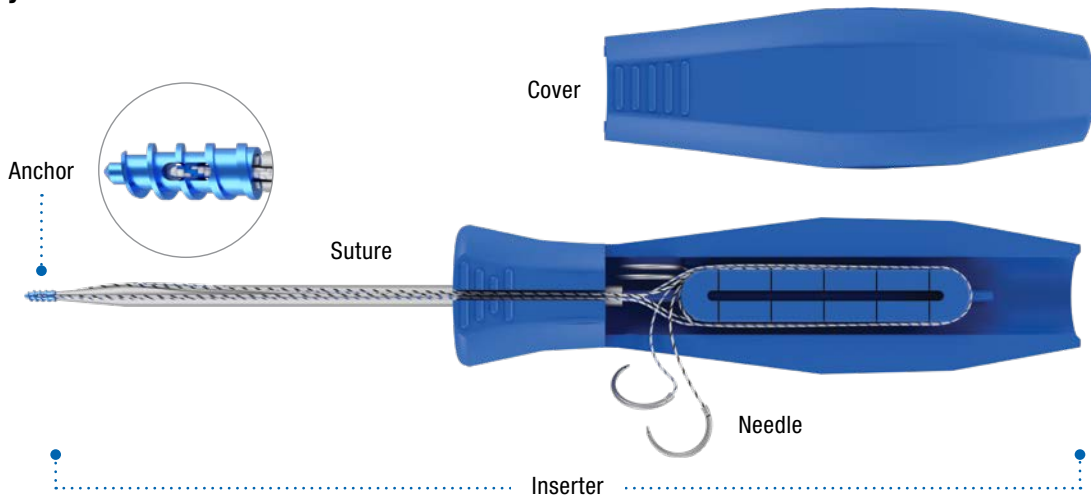

Drill



Manufactured by: Ortho-Design (PTY) Ltd. / 17 Dely Road / Hazelwood, Pretoria / South Africa / (+27) 12 807 5172
www.ortho-design.com

Distributed by: TriMed, Inc. / 27533 Avenue Hopkins / Valencia, CA 91355 USA / 800-633-7221 / www.trimedortho.com

The technique presented is one suggested surgical technique. The decision to use a specific implant and the surgical technique must be based on sound medical judgment by the surgeon that takes into consideration factors such as the circumstances and configuration of the injury.

This document is controlled by TriMed, Inc. When downloaded, printed, and/or copied, this document becomes uncontrolled, and users should always check trimedortho.com for the latest version.

² For indications, contraindications, warnings and precautions related to the TriMed Suture Anchors refer to IFU on trimedortho.com/ifu.

